

## INTENDED USE

wheeze® is intended to monitor wheeze, an abnormal breath sound and sign of airflow limitation in the lungs. It is the sound the doctor listens for with a stethoscope when examining a patient during an asthma flare-up.

wheeze is intended to monitor


wheeze in adults and children over 2 years with diagnosed asthma. Adults should assist children under 14 years old. Regular use of wheeze during daily activities will help inform patients about whether their wheeze rate is increasing, decreasing or remaining stable.

## WHAT'S IN THE BOX

wheeze Breath Sensor

USB-A to USB-C charging cable

Quick Start Guide

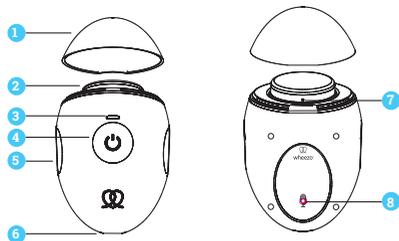

1. Protective cap
2. Sensor diaphragm (applied part)
3. LED indicator
4. Power button
5. Finger grips
6. USB charging port
7. Air vent
8. Background microphone

## SMART DEVICE COMPATIBILITY

Owner supplied smart device

Made for iPhone 6s and up running iOS 13 or later

Select Android devices

## SETTING UP wheeze FOR THE FIRST TIME

Download the wheeze app



Connect via Bluetooth®

## wheeze STEP-BY-STEP SET-UP

1. wheeze breath sensor is powered by a built-in rechargeable battery. Before using for the first time, plug one end of the micro USB charging cable supplied into the end of the device and the other end into a compliant AC adaptor. Charge it for up to three hours before first use. A fully charged battery typically lasts up to 20 recordings.
2. While charging, the breath sensor's LED indicator will flash green. When it is fully charged, the LED will display solid green.
3. Download the wheeze app from the App Store or Google Play and launch it to connect the breath sensor to the app.
4. Enable Bluetooth® on your mobile device. Allow the app to access and use Bluetooth® and follow the instructions on the screen. Note: Android requires Location Services to be enabled for the initial Bluetooth® pairing.
5. Remove wheeze breath sensor's protective cap and turn it on by holding the power button down until the LED indicator flashes blue indicating it is ready to pair with your smart device.
6. When the wheeze breath sensor and your smart device are paired, the LED indicator will display solid blue.
7. If you require assistance to set up your wheeze, please contact support@wheeze.com.

## LIMITATIONS AND CONTRAINDICATIONS

wheeze is a home monitoring device not intended for diagnostic use. When wheeze is used to monitor wheezing in lung conditions such as asthma, the user should be under the care of a licensed healthcare professional.

The most common causes of recurrent wheezing are asthma and chronic obstructive pulmonary disease (COPD), however, any inflammation in the patient's throat or larger airways can cause wheezing. Common causes include infection, an allergic reaction or a physical

obstruction such as a tumour or a foreign object that has been inhaled.

A licensed healthcare professional's advice is required to understand the meaning and importance of wheeze rate.

**Vocal cord dysfunction (VCD).** wheeze may not be suitable for use by people diagnosed with VCD. VCD causing audible wheeze may interfere with the device's ability to analyse breath sounds for wheeze rate associated with lung conditions.

## OPERATING ENVIRONMENT

wheeze has been designed for use in a quiet environment e.g. home, school or office. It is not designed to be used in direct air currents (e.g. wind), source of heat or cold, direct sun rays or

other source of light or energy, dust, sand or any other chemical substances or performance will be affected.

## WARNINGS

If the user is feeling breathless, or experiencing other symptoms, it is important to prioritise treatment over taking a wheeze rate. Do not delay treatment.

Regardless of the wheeze rate, if the user has signs and symptoms such as chest tightness, shortness of breath,

coughing or wheezing, they should always follow their healthcare professional's advice.

In a severe asthma attack, there may be no wheeze. This is called 'silent chest' and may be life-threatening.

## IMPORTANT SAFETY INSTRUCTIONS

Once removed from its packaging, check that there is no visible damage on the wheeze breath sensor. In the case of damage, do not use it and return it to the seller.

Use only with iOS and Android devices specified on wheeze.com.

Use only an AC adaptor that complies with IEC 60950-1 or IEC 60601-1 for charging. The micro USB port on the device is only for charging of the internal battery.

Keep the USB charging cable provided out of the reach of children.

Use only in environments specified in the full Owner's Manual.

Do not expose to liquids.

Do not tamper with the blue sensor diaphragm. Degraded sensors and microphones can affect performance.

Do not expose the device to

extreme temperatures - refer to full owners manual.

Do not drop the device or treat it roughly in any way.

Never attempt to repair the device yourself. Opening of wheeze invalidates all guarantee claims.

The normal function of the product may be disturbed by strong Electro-Magnetic Interference. If so, simply reset the product to resume normal operation by following the instruction manual. In case the function cannot resume, please use the product in a different location.

In compliance with the European Directive on Waste Electrical and Electronic Equipment (WEEE) 2012/19/EU, do not dispose of this product as unsorted municipal waste. Follow local, state and national governing ordinances and recycling instructions reading disposal or recycling of the device.



# RESPIRI



Manufactured for and Distributed by:  
Respiri Limited

9/432 St Kilda Road, Melbourne, VIC, 3004, Australia

For additional information:

✉ info@respiri.co

1300 11 RESP

1300 11 7377

DESIGNED IN AUSTRALIA



Authorised European Representative  
Compliance Management Solutions  
Europe

UK Responsible Person  
European Device Solutions Ltd

2 Bulgaria Str. 2795  
Petrich Bulgaria

15 Coanwood Drive, Whitley Park, Tyne &  
Wear NE25 9GB, UK



wheezeo®

and wheezeo® are registered trademarks of Respiri Limited

## LIMITED WARRANTY AND RETURNS POLICY

### WHEEZO LIMITED WARRANTY AND RETURNS POLICY

This Limited Warranty and Returns Policy ("Limited Warranty") applies to the wheezeo product, including accessories (the "Product") and is given by Respiri Limited to the individual or entity (the "Owner") who uses the Product which has been obtained from Respiri or a Respiri authorised dealer. This Limited Warranty applies only within countries where the wheezeo app is available for customers to download and for customers located in such countries, and is not transferable.

### APPLICATION OF AUSTRALIAN CONSUMER LAW

The following paragraph applies in Australia only.

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.

To the extent permitted by statute the liability, if any, of Respiri arising from any failure to comply with a statutory guarantee under the Australian Consumer Law, in relation to the supply of goods or services other than goods or services of a kind ordinarily acquired for personal, domestic or household use or consumption, is at Respiri's option limited to, in the case of goods:

- the replacement of the goods or the supply of equivalent goods;
- the repair of the goods;
- the payment of the cost of replacing the goods or of acquiring equivalent goods; or
- the payment of the cost of having the goods repaired, and in the case of services:
- the supplying of the services again; or
- the payment of the cost of having the services supplied again.

### 30-DAY RETURN POLICY

If you purchased the Product directly from Respiri, then you may return the Product within 30 days of receiving it, in order for a refund under this Return Policy to be granted, you must request a Return Merchandise Authorisation (RMA) number from Respiri during the 30-day period following the date you received the Product from Respiri. You may request a RMA number by submitting a request to support@respiri.co. The Product must be received in "as new" condition by Respiri within 3 weeks from the date on which the RMA was issued in order for you to receive a refund.

If the Product is returned to Respiri in accordance with this Return Policy, Respiri will credit the account you used to make the original purchase with the full purchase price you paid for the Product and any applicable taxes minus any shipping and handling charges you paid when you purchased the Product. You are responsible for and must prepay all return shipping charges and you assume all risk of loss or damage to the

Product while in transit back to Respiri.

The time to credit your account may vary, depending on processing time. Please allow up to seven business days for Respiri to credit the account used to make the original purchase. If you received a damaged Product directly from Respiri, please contact support@respiri.co.

Purchases through an Authorised Dealer: If you purchased the Product from a Respiri authorised retailer or distributor, please contact that retailer or distributor. You must follow the return policy of that authorised retailer or distributor.

### MANUFACTURER'S LIMITED WARRANTY

The benefits to the Owner given by the warranty stated below are in addition to other rights and remedies the Owner may have under the Australian Consumer Law (and any other applicable law) in relation to the Product.

Respiri warrants to the Owner that the Product will materially conform to its published specifications and be free from defects in materials and workmanship under normal use for a period of the greater of 1 year or the amount of time required by applicable consumer regulations in the country of purchase and commencing on the date of delivery (the "Warranty Period").

If the Product proves defective during the Warranty Period, you must follow the instructions set out below for returning the Product, or part thereof, and Respiri will replace the Product with a product that is the same or equivalent to the Product you purchased.

A replacement Product assumes the remaining warranty of the original Product, or 30 days from the date of replacement, whichever is longer. Respiri reserves the right to replace any discontinued Product with a new product of comparable value and function.

### HOW TO OBTAIN WARRANTY SERVICE

To obtain warranty service on a Product purchased directly from Respiri, contact support@respiri.co.

Respiri may require proof that you bought the Product directly from Respiri and of the date of purchase.

If Respiri determines that a Product is eligible for warranty replacement,

Respiri shall bear the costs associated with returning a Product under warranty. The Owner can claim these costs from Respiri by providing evidence of charges along with a refund request via email to support@respiri.co. Respiri is not responsible for items damaged or lost in transit. Replaced Products are the property of Respiri. The time to credit your account may vary, depending on processing time.

Please allow up to seven business days for Respiri to credit the account used to make the original purchase. If you received a damaged Product directly from Respiri, please contact support@respiri.co.

Purchases through an Authorised Dealer: If you purchased the Product from a Respiri authorised retailer or distributor, please contact that retailer or distributor. You must follow the return policy of that authorised retailer or distributor.

A replacement Product provided under this Limited Warranty will only be dispatched to addresses within countries where the wheezeo app is available for customers to download and for customers located in such countries, and is not transferable.

Respiri covers the Product for the Warranty Period as long as proof of purchase can be provided, please contact support@respiri.co.

### WHAT IS NOT COVERED?

Without limiting any rights and remedies the Owner may have under the Australian Consumer Law (and any other applicable law) in relation to the Product, this Limited Warranty will be void if the Product: (a) is used in any manner that is inconsistent with the intended purpose or design of the Product as described in Respiri's published documentation for the Product, (b) is altered in any way, including being opened, dismantled or otherwise disassembled, (c) is used or maintained in any manner that is inconsistent with Respiri's instructions or warnings included with the Product or (d) is subjected to any other misuse, abuse, neglect, accident, or external forces.

This Limited Warranty does not cover normal wear and tear.

Respiri does not warrant that the operation of the Product will be uninterrupted or error-free.

**WARRANTY DISCLAIMER:** To the greatest extent permitted by applicable law, this Limited Warranty replaces all other warranties, conditions, guarantees and liabilities whether oral, written, (non-mandatory) statutory, contractual, in tort or otherwise, including without limitation, and where permitted by applicable law, any implied conditions, guarantees, warranties or other terms as to satisfactory quality or fitness for purpose.

To the greatest extent permitted by applicable law, the warranties set forth in this limited warranty are exclusive and no other warranty, whether written or oral, expressed or implied, applies with respect to THE PRODUCT OR ANY SERVICES PROVIDED IN CONNECTION WITH THE PRODUCT, INCLUDING WITHOUT LIMITATION ANY WARRANTY OF SUITABLE DESIGN, MERCHANTABILITY, SATISFACTORY QUALITY, FITNESS FOR A PARTICULAR PURPOSE (EVEN IF RESPIRI HAS BEEN INFORMED OF SUCH PURPOSE) OR AGAINST INTELLECTUAL PROPERTY INFRINGEMENT.

IF ANY IMPLIED WARRANTY CANNOT BE DISCLAIMED UNDER APPLICABLE LAW, THEN SUCH IMPLIED WARRANTY SHALL BE LIMITED IN DURATION TO THE WARRANTY PERIOD SET FORTH ABOVE.

### LIMITATION OF REMEDIES

SUBJECT TO ANY RIGHTS AND REMEDIES THE OWNER MAY HAVE UNDER THE AUSTRALIAN CONSUMER LAW (AND ANY OTHER APPLICABLE LAW) IN RELATION TO THE PRODUCT, THE REMEDIES IN THIS LIMITED WARRANTY ARE YOUR SOLE AND EXCLUSIVE REMEDIES IN CONNECTION WITH ANY DEFECT IN OR PROBLEM WITH THE PRODUCT, EVEN IF RESPIRI HAS BEEN NEGLIGENT.

Resellers, retailers, distributors, agents, or

employees of Respiri are not authorised to make any modification, extension, or addition to this Limited Warranty.

To the greatest extent permitted by applicable law, Respiri does not assume any liability for loss of or damage to or corruption of data, for any loss of profit, loss of use of the Product or functionality, loss of business, damage to reputation, loss of contracts, loss of revenues or loss of anticipated savings, for any increased costs or expenses or for any indirect loss or damage, consequential loss or damage or special loss or damage.

To the greatest extent permitted by applicable law, Respiri's liability in connection with any To the greatest extent permitted by applicable law, Respiri's liability in connection with any defect in or problem with the Product shall be limited to the purchase value of the Product.

THE LIMITATIONS SET FORTH IN THIS LIMITED WARRANTY ARE INTENDED TO LIMIT THE LIABILITY OF RESPIRI AND SHALL APPLY NOTWITHSTANDING ANY FAILURE OF ESSENTIAL PURPOSE OF ANY LIMITED REMEDY.

### OTHER IMPORTANT NOTICES

The rights provided to you under this Limited Warranty are provided in addition to all rights and remedies conveyed to you by applicable consumer protection laws and do not affect your legal (statutory) rights under your applicable national law.

The jurisdiction applicable to you may not allow the limitations of liability or damages set forth in this Limited Warranty, in which case such limitations shall only apply to you to the extent permitted in such jurisdiction.

Your use of the wheezeo app is subject to Respiri's App Terms and Conditions, which may change from time to time. You can find the latest version on our website at wheezeo.com/app/terms/ in the case of any inconsistencies between the App Terms and Conditions and this Limited Warranty, this Limited Warranty will prevail.

This Limited Warranty does not apply to any mobile device or the cellular and/ or other network or system on which the Product (together with the wheezeo app) operates. Therefore, RESPIRI WILL NOT ACCEPT ANY RESPONSIBILITY UNDER THIS LIMITED WARRANTY for the operation, availability, coverage, services or range of the cellular or other network or system.

Always completely follow safety, use, operation, and maintenance instructions for the Product or any other Respiri product.

This Limited Warranty is governed by the laws of Victoria, Australia.



# wheezeo®



## QUICK START GUIDE

# Digital Wheeze Monitor Model WZO01